



Problem solving

Simple solutions to common problems



MA.9

Introduction

1

In this session participants will:

- Look at simple solutions to some of the common fitting problems caused by limitations in the sizes and range of wheelchairs available
- Note that making modifications and providing more postural support is covered in much more detail in the intermediate level training.

MA.9

Common fitting problems


2

- What are some common fitting problems that you have already seen, or think may happen?
- Common fitting problems include:
 - Seat depth is too long or too short
 - Footrests height cannot go up high enough or down low enough
 - Legs tend to roll inwards or outwards
 - Feet tend to slide off footrests
 - Wheelchair is too wide.

MA.9 3

Problem: seat depth is too short

- Lengthen the wheelchair seat depth by extending the seat rails and replacing upholstery.
- Lengthen the wheelchair seat depth by replacing the upholstery.
- Lengthen the wheelchair seat depth by adding a rigid board with a cushion.

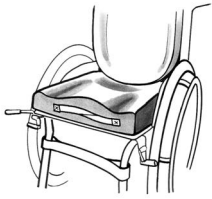

MA.9 4


Caution: Replacing a wheelchair seat

- When replacing a wheelchair seat, make sure the new seat is strong enough to carry the weight of the person.
- If it is a rigid seat:
 - Check that the seat does not flex or crack beyond where it extends past the existing wheelchair seat
 - Check during fitting that the backrest and footrests are the correct height. A solid seat and cushion will raise the person in the wheelchair.

MA.9 5

Caution: Replacing a wheelchair seat (continued)

- When cutting a rigid seat – always ensure that all edges are smooth and splinter free.
- When cutting a cushion – always cut from the front so that pressure relief at the back of the cushion stays the same.

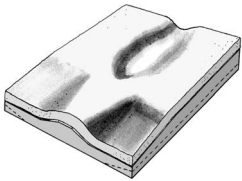

MA.9 **Problem: seat depth is too long** 6

- Mark the seat depth that is needed on the existing seat and reduce it.
- If the seat is upholstered – remove the upholstery, shorten it (using an industrial sewing machine) and replace.
- If the seat is rigid – remove the seat, shorten and replace.
- Shorten the cushion to match the new seat depth.


MA.9 **Caution: When cutting** 7

Remember:

- When cutting a rigid seat – always ensure that all edges are smooth and splinter free
- When cutting a cushion – always cut from the front so that pressure relief at the back of the cushion stays the same.

MA.9 **Problem: footrests are too low** 8

- Raise the footrests height by shortening the footrest hangers.
- Raise the footrests height by building up the footrests.

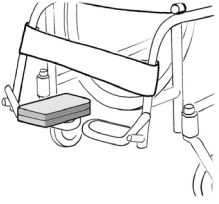

MA.9 9


Caution: When changing footrest hangers

- When changing the footrest hangers – always check that the adjustment mechanism still works.
- If building up footrests – check that the build-up does not stop footrests that move out of the way from working.

MA.9 10

Problem: footrests are too high

- Lower the footrests by lengthening the inner extension tube.
- Increase the height of the cushion.

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


MA.9 11

Caution: When lowering footrests


- If lowering the footrests - make sure the footrests do not become so low that they catch on the ground.
- If raising the height of a cushion:
 - Do not use soft foam to increase the height of the cushion
 - Check during fitting that the backrest and armrests are the correct height
 - Check that there are not functional problems with raising the cushion.

MA.9 12
Problem solving fit or adjustment - Gary




MA.9 13
Legs tend to roll inwards or outwards

- Double check the footrests height
- If the seat is canvas, check that the seat is properly tensioned
- Add wedges to the cushion to support the thighs in neutral, or provide a contoured cushion.

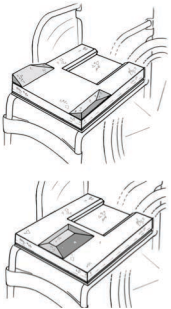

MA.9 14
Problem: feet tend to slide off footrests

- Double check the footrests height
- Adjust the footrests angle
- Add a strap.

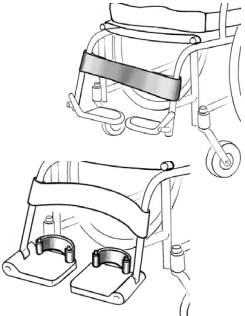


MA.9 15

Caution: When altering footrests

- Whenever altering the footrests:
 - Try to ensure there is even pressure under the foot.
- When adding a strap:
 - Make sure the strap is easy to remove so that the person can get in and out of the wheelchair without effort
 - Check that the person can reach the strap without assistance, and can do it up and undo it easily.

MA.9 16


Problem solving fit or adjustment - Arun



MA.9 17

Problem: wheelchair is too wide

To reduce wheelchair width, add foam inserts on each side of the pelvis.

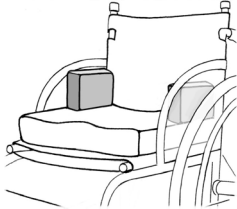



Caution

18

- The pads only need to come as far forward as the person's trunk.
- Ensure the cushion matches the wheelchair seat width.
- If the wheelchair is very wide, the person will also have trouble reaching the push rims.
- Over-reaching for the push rims can cause a shoulder injury. Consider carefully whether the wheelchair is safe for the person before prescribing it.



Stump boards

19


- Useful for clients with a transtibial (below knee amputation) who do not have a prosthesis.
- Benefits:
 - Prevents knee contractures post amputation
 - Protects the residual limb from injury
 - Prevents swelling of the residual limb.





How to make a stump board

20

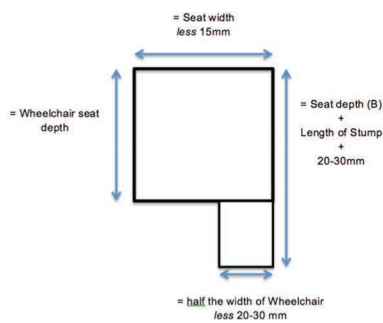



Key point summary

21

- Problems with the wheelchair fit may be due to:
 - Limitations in the sizes and range of wheelchairs available
 - The person needing extra support to be able to sit upright comfortably.
- Simple solutions can sometimes solve these problems.
- Before making any modifications, always check you have the best size of the wheelchair available for the person, and it is adjusted as well as it can be.
- Whenever making any modifications:
 - Recheck the wheelchair to make sure it is safe to use
 - Carry out a complete fitting again with the person, as changes to one thing can affect others.



Thank you



This training material was developed by Motivation Australia, and builds upon World Health Organisation material. Funding was provided by the Australian Government through the ANCP programme.



Copyright

Copyright is retained by the Authors. However free permission is granted to republish or reproduce this document for non-commercial use. If part or whole of this document is republished or reproduced in any form it must be accompanied by an acknowledgement of Motivation Australia and include a reference for this document.

Where copyrighted material from a third party has been used in this document and acknowledged by Motivation Australia, permission should be sought from the original copyright holder to republish or reproduce their work.

Permission should be sought from Motivation Australia before any illustrations, graphics, photographs or any other type of image is used separately from the rest of this document. For more information contact info@motivation.org.au.

www.motivation.org.au
